


Cleaning instructions for the 3102-OXY-HB4, nut with a tubing nipple

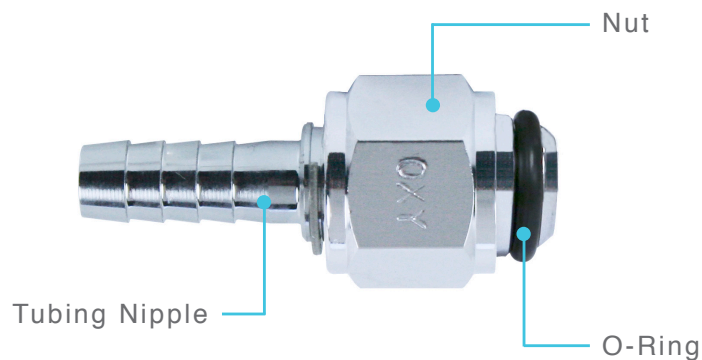

The nut and tubing nipple are made of nickel-plated brass. The adapters are meant for multiple uses, not single patient use. If you would like to sterilize the nut and tubing nipple, please remove the O-ring from the tubing nipple. The reusable nut and tubing nipple are primarily used with oxygen & air flowmeters, and can be wiped down. They can also be cleaned with soap detergent & water as well as PDI wipes, BPI wipes, Hypo-chloride wipes and Quaternary wipes.