


Flush the Unit in Each Functional Mode

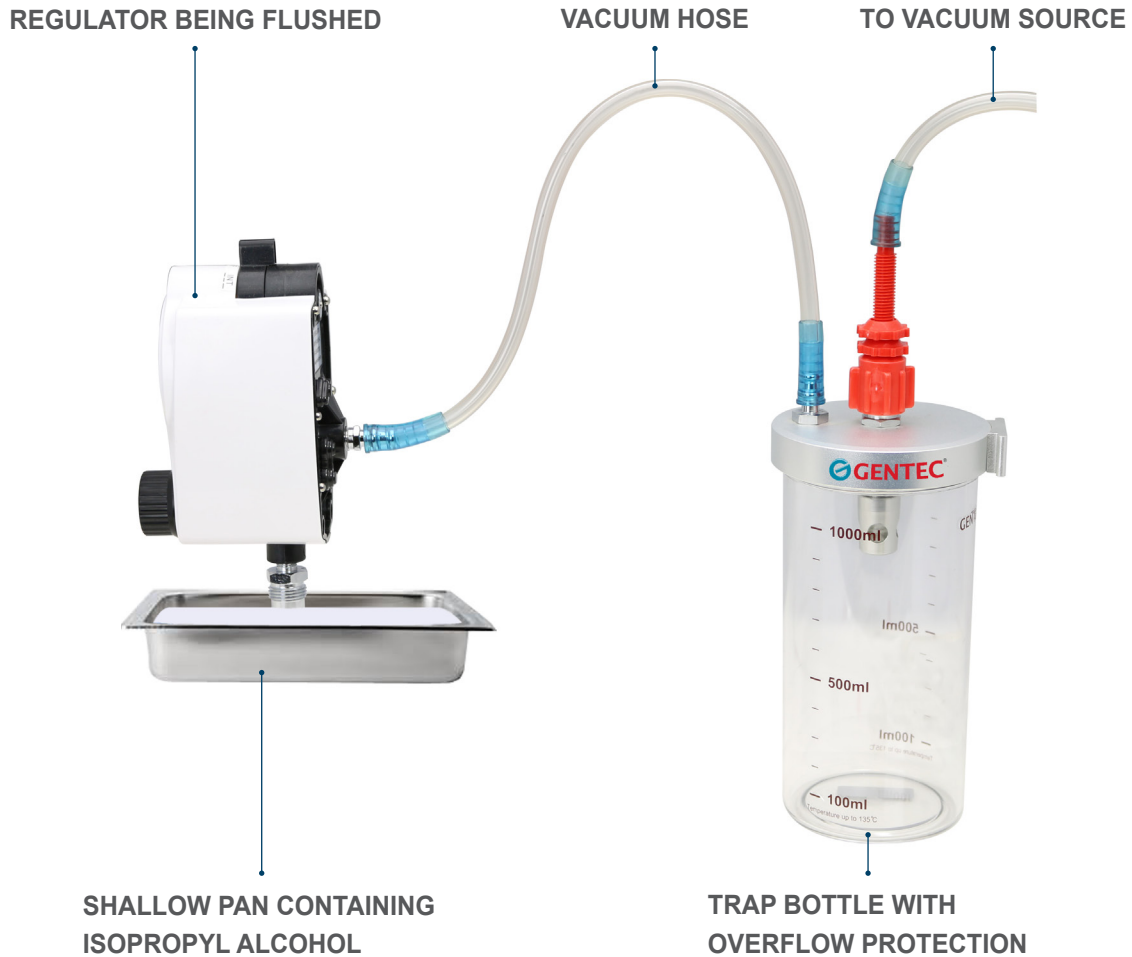

1 Turn the mode selector switch to OFF. Attach two vacuum hoses as shown above. The collection bottle will trap the liquid and keep it out of the hospital vacuum system. Place the patient connection of the GENTEC® regulator into a shallow pan containing isopropyl alcohol.

2 Turn the mode selector switch to the desired mode and turn the vacuum level knob (lower range knob) clockwise as

far as it will go for maximum flow. Draw approximately 100 ml of isopropyl alcohol through the unit.

3 Remove the unit from the alcohol container and operate the unit for at least 30 minutes with the patient connection of the regulator open to the atmosphere. This allows the internal passages to dry out.

4 Test the unit.