



## Vacuum Safety Trap Bottle Operating Instructions



880VT-1



880VT-3



880VT-5

## Save These Instructions

Genstar Technologies  
4525 Edison Ave., Chino, CA. 91710  
[www.gentec.com](http://www.gentec.com) • 909.606.2726

 **CAUTION**

Federal (USA) law restricts this device to sale by or on the order of a physician.

## Intended Use

To protect suction regulators and central vacuum systems from liquid contamination.

## Read All Instructions Before Using

This manual instructs a Professional to install and operate the Vac Trap. This is provided for your safety and to prevent damage to the Vac Trap.



### **WARNING**

Use this Vac Trap only for its “Intended Use” as described in this manual.



### **CAUTION**

- Only personnel instructed and trained in its use should operate this Vac Trap.
- Be sure all connections are tight and leak free.



## PARTS LIST

	Description	Part Number
A	Float Assembly	880VR-306
B	Jar	880VR-308
C	Tubing Nipple	880VR-304

### CAUTION

The Vac Trap must be in the vertical position to function.

## TROUBLESHOOTING

Problem	Probable Cause	Remedy
Float Stuck	1. Float neck dirty	1. Remove foreign substance
No Suction	1. Leaking connections 2. Suction Source not turned "ON" 3. Jar full	1.1 Check connections and reconnect 1.2 Tighten Jar 2. Turn Suction Source "ON" 3. Empty/clean jar

## Cleaning

You can choose between these two types of cleaning. Cleaning agent or sterilization.

1. For the jar: (soap detergents, Cavicide spray or wipes, or Isopropyl Alcohol)
2. Steam sterilization: 3-5 minutes at 135°F or 15-20 minutes at 135°F (for both)

### CAUTION

- Exceeding the recommended sterilization temperatures may shorten the life of the Vac Trap.
- DO NOT autoclave.

## Maintenance

Before each use, inspect the Vac Trap for any damage; DO NOT USE if damaged.

## Disposal instructions

Dispose of the Vac Trap in accordance with the local regulations.

Please Recycle 

### WARNING

Dispose of the Vac Trap using Biohazard precautions.



**Genstar Technologies Co., Inc.**  
**4525 Edison Avenue**  
**Chino, CA 91710 USA**  
**TEL: 909.606.2726**  
**FAX: 909.606.6485**  
**[www.gentec.com](http://www.gentec.com)**